Elasto-Gel™ Cast and Splint Padding

STERILE R

CS5500

4" x 4" • 10cm x 10cm

CS5600

6" x 8" • 15cm x 20cm

CS5700

12" x 12" • 30cm x 30cm

CS5800

8" x 16" • 20cm x 40cm

Made in the USA

Manufactured By:

southwest technologies inc.

1746 Levee Rd., North Kansas City, MO 64116
Phone: (816) 221-2442 • Toll Free: (800) 247-9951
Fax: (816) 221-3995 • email: info@elastogel.com

© SWT# 9-860-003 Rev. E 8-10

M. Devices Group
Marborough House
Riding Street • Southport • U.K.
Phone: +44 1704 544 944
Fax: +44 1704 544 050

Distributed By:

www.elastogel.com

On West No. 4th

Call: 1-800-247-9951

Southwest Technologies

Products available from

*Therapy Products and Wound Care
Fole about other advanced Hot/Cold*

(English)

The **Elasto-Gel™** Cast and Splint Padding is designed to protect skin from friction, protect bony prominences from pressure or to pad devices to provide a soft surface. The product is easy to use and comfortable for the patient. These products are used extensively under pressure garments, which are used for scar management.

DIRECTIONS FOR USE

1. If using under a cast, all traces of soaps, detergents, and disinfectants should be removed from the skin before applying the padding.
2. Select a cast/splint pad of appropriate size and remove from the package.
3. The pad may be cut to size and/or shape with scissors. Leave the clear plastic film on gel while cutting the padding. Holes may be cut in the pads to fit over and between fingers, toes, and other protrusions to allow for complete protection of the target area.
4. Remove clear plastic film from gel and position the pad and proceed with the application of the cast, splint, or put on the pressure garment. Alternatively, the cover roll may be used first, then gel, then apply cast.
5. If a splint needs to be stabilized then the white fabric backing may be removed. To do so, moisten the fabric (using a damp cotton swab). Slowly "peel" the cloth from the gel and apply the gel pad to the patient. Then remove the clear plastic film, put the splint in place, and secure it as needed.
6. **Elasto-Gel™** Cast and Splint Pads may be layered on top of each other for additional padding.

Caution: DO NOT get the cast and splint padding wet.

Caution: If skin irritation occurs, discontinue cast and splint pad use.

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