

# SW-9004 Small/Medium Shoulder Sleeve

## SW-9005 Large/X-Large Shoulder Sleeve

**\*CAUTION: DO NOT OVERHEAT!!** Due to the nature of the gel, it may only feel warm to the touch, but if the sleeve has been overheated, too much heat may be transferred to the body and can cause burns.

### The ideal hot/cold therapy.

The **Elasto-Gel™** Therapy Products are designed to provide **moist heat** therapy or **soothing cold** therapy for relief of minor muscular and joint aches and pains. The hot or cold therapy will last 20 to 40 minutes per application. The product is held in place with hook and loop type closures, which allow the user to move about during therapy. The product is microwavable so it can provide nearly continuous heat therapy. When used cold, the product conforms to contours, even at -20°F (-30°C), allowing comfortable soothing cold applications.

### A specially formulated gel:

This product is made with **Elasto-Gel™**, a patented\* gel with exceptional properties for transferring heat or soothing cold. If punctured, it will not leak and will remain usable. The unique combination of specially selected fabric and tough flexible gel provides an exceptional durable product for use in hospitals, rehabilitation centers, athletic facilities and the home.

## DIRECTIONS FOR USE

### 1. COLD THERAPY:

Store the sleeve in the freezer for a minimum of two (2) hours. It may be left in the freezer indefinitely until ready to use. It will remain flexible even at -20°F. Remove the sleeve prior to defrosting the freezer. **Note: Always store product in a plastic bag while in freezer.**

### 2. HEAT THERAPY:

The most convenient method of heating the sleeve is in a **microwave oven**. Suggesting heating times: **600-700 watt microwave:** 60-90 Seconds. Turn the sleeve inside to out and re-heat for 60-90 Seconds. Then refold the sleeve and heat for 45 Seconds. **1100 watt:** 50 Seconds per side.

Wait one minute (60 seconds) before using. The sleeve should feel **WARM** to the touch, **NOT HOT**. Hold the sleeve firmly against the skin for 30 seconds **before re-heating\***. If the sleeve is not warm enough, heat in 20 second increments and rotate the product each time.

The sleeve can also be heated in a **conventional oven** at 250°F for 15-20 minutes. Place sleeve on cookie sheet or baking dish in the center of the oven. Do not allow the straps or buckles to touch the heating element. Rotate the sleeve after 6 minutes, top to bottom and inside to out.

### 3. DO NOT GET THE PRODUCTS WET:

The gel will absorb water and swell. If the product becomes damp, lay it on a flat surface and allow it to fully air dry before continuing use. Do not use wet towels under the gel.

### 4. CLEANING INSTRUCTIONS:

To clean the sleeve, wipe the cover down with a damp (not wet) cloth using mild soap or disinfectant. Allow the product to fully air dry before continuing to use.



Treating the World Well®  
**southwest technologies, inc.**

1746 Levee Road • North Kansas City • MO 64116

Phone: (816) 221-2442 • Toll Free: (800) 247-9951

Fax: (816) 221-3995 • e-mail: info@elastogel.com

Visit our Website: www.elastogel.com

MANUFACTURER



**EU Authorized Representative:**

**M. Devices Group**

Marlborough House • Riding Street

Southport • PR8 1EW • UK

Phone: 00 44 (0) 1704 544 944

Fax: 00 44 (0) 1704 544 050

e-mail: mdg@tour.demon.co.uk