Spanish

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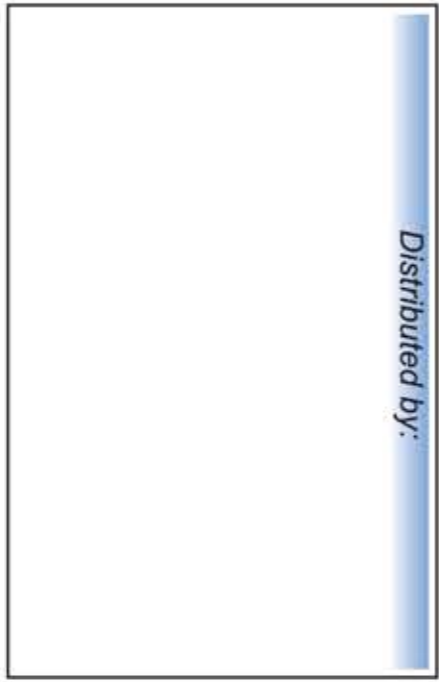
southwest technologies inc.

# Elasto-Gel™

DON'T HIDE  
BEHIND A  
MASK!  
REJUVENATE  
FROM IT!

Treating the world well®

southwest technologies inc.

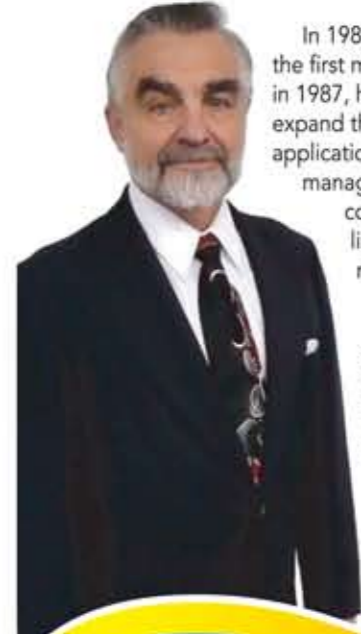
## Quality Policy

It is the policy of Southwest Technologies Inc. to design, manufacture, market and distribute products consistent with our high quality standards. We periodically review the products for design and performance improvements.

The products will be designed and manufactured to reliably perform their intended functions as labeled. All products will be produced and marketed within the established company guidelines.

## Dr. Edward I. Stout PhD

Dr. Stout received his MS in Chemistry from Bradley University 1968. He obtained his PhD in Organic Chemistry, with expertise in Polymer Chemistry, from the University of Arizona in 1973. He has positively impacted the world through the development of his patented unique Gel Technology now known as; Elasto-Gel.™ Dr. Stout founded Southwest Technologies Inc. in 1980 with the desire and vision to "treat the world well" by providing patients comfort and improved healing.

In 1983 Dr. Stout developed and marketed the first microwaveable/dual Hot/Cold pack and in 1987, he received the FDA approval to expand the Elasto-Gel™ technology for applications in wound management, scar management and padding materials. He continues to develop innovative product lines with the patient's well being in mind.

Dr. Stout has authored and published numerous scientific papers and holds many patents. He has been nominated for the Who's Who for the last twenty years both domestically and internationally and won the prestigious award of International Man of the Year 2001 for his developments in wound care products.

## Resurfacing Application Instructions

To obtain the best healing results apply the Elasto-Gel™ Face Dressing\* immediately after surgery while patients are still lying on the table.

1. Remove the Elasto-Gel™ Face Dressing from the pouch.
2. Peel the top clear release film off of the Elasto-Gel™ dressing.
3. Gently place the Elasto-Gel™ Face dressing on the face and remove the second release film.
4. Align the mouth, eyes, and nose openings. Take special care to ensure eyelash freedom.
5. Remove the white cloth cover from the package and place it over the Elasto-Gel™ Face Dressing then secure with tie straps.
6. The first dressing change should occur after 8 to 24 hours. Apply the second dressing by using techniques similar to the first application. This second dressing may remain in place 2 to 3 days. Use a third dressing if necessary.
7. Additional dressings may be required for deeper resurfacing

- or surgical procedures.
8. After wound care closure, applying moisturizing agents may improve comfort and the healing outcome. We recommend Stimulen™ Moisturizing Lotion.

\* Clinical experiences indicate that application of a cooled dressing immediately after facial or laser surgery helps reduce thermal inertia and results in less tissue damage. Similar results have been reported for thermal burns and radiation burns.

\*\* Cutting the dressing is easier if the clear release film is kept in place. We suggest that cutting and shaping be completed as much as possible before applying the dressing to the patient.

**Precautions:** 1. Edema is a natural consequence of any type of facial resurfacing. 2. Excessive swelling around the mouth and eyes can occur. The patient should be instructed to lie down every three to four hours and rest for thirty minutes before returning to normal routine. 3. Men may experience itching on the 2nd or 3rd day. 4. Do not get the dressing wet. 5. Avoid wearing the dressing in the shower. 6. Consult a physician if signs of infection occur – redness, excessive swelling, fever, etc.

## Ordering Information

Stock #	Description	Packaging
FM7500	Elasto-Gel face mask dressing	2 gel face masks 2 outside covers per box

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© Rev. C-6-09USA SBT# 9-960-099